



Installation in 4 steps

ARKY indoor cabinet



Step 1

Start by holding the cabinet in the position where it is to be mounted. Make sure that the cabinet is level.

Open the door of the cabinet and mark with a pencil or pen the top two holes in the cabinet.

Remove the cabinet and drill the marked holes with a fitting drill for the wall and supplied wall plugs.



Step 2

Insert the wall plugs into the drilled holes and hold the cabinet in position against the wall. Mount the cabinet using the supplied screws and rings (make sure the rings are mounted on the inside of the cabinet).

Once the cabinet is mounted to the wall remove the cord handle and mount the supplied aluminum handle with the supplied screws.



Step 3

The cabinet is delivered with a protective foil on the window to prevent damage from transportation or installation. This foil is attached to the window on the inside of the cabinet. Peel away this foil from the bottom right hand corner.



Step 4

The cabinets 60.112 and 60.122 need to have the alarm activated before use. The on/ off switch is located on the right hand side of the alarm unit. You can find the alarm in the top right hand corner of the cabinet. Once activated the alarm will sound until the door of the cabinet is fully closed.

